

CLEANING ANNIE AND BRAYDEN CPR MANIKINS

It is important to give our manikins a good clean with warm soapy water every 6-12 months (depending on amount of use). The faces from the Annie dolls can either be put on a hot wash in the dishwasher or wiped.

It would be wonderful if you are happy to do this to the kit you hold – if not, please arrange with us to get it back to Helimed House so we can clean it for you.

If you are happy to clean your Annie yourself, we've got some helpful photos to demonstrate how to dismantle the dolls (and therefore put them back together!)

Thanks so much if you're happy to give it a go!



1. Annie's face will become unattached if you lift both her ears off over the clips.



2. Her mouthpiece will come unattached like this.



3. Wipe with a damp cloth or sponge around her head/hair.



4. Top tip! Use a sponge with a scourer to give the torso a good clean, as the wipes can leave a film and a bit of elbow grease is needed to remove this and any marks.




5. To remove the torso skin, release Annie's two neck and two side clips.



6. Pull the skin back to clean the inside of the torso. If the doll has this blue patch, please be careful to not use too much water, and squeeze the sponge out well before cleaning.




7. If your doll has lungs, these can be released and moved out of the way to clean the top of the rib cage.

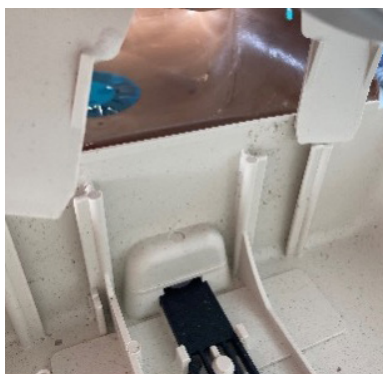


8. You can unclip the rib section at the bottom, and at the neck, to remove the rib section completely.



9. Please use a damp cloth to wipe inside - including all the nooks and crannies!

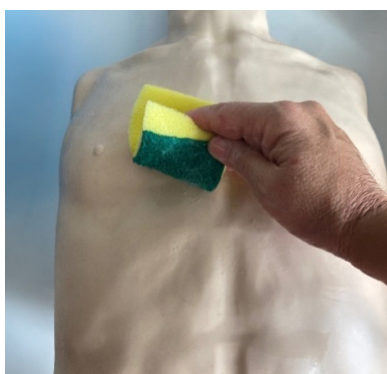

10. Brayden can be cleaned in the same way - but be sure to not use too much water due to the electrics which make his lights work.

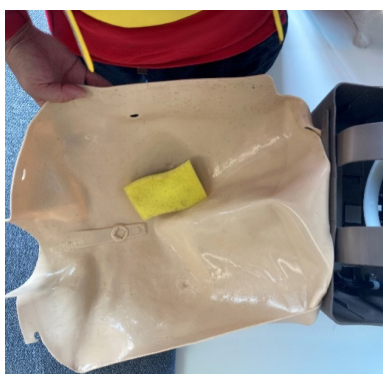

11. In our experience, Brayden doesn't seem to get as dirty inside as Annie - this may be due to his skin being held in place more tightly.



12. Brayden's face can be removed in the same way as Annie's but shouldn't be put in the dishwasher. Please clean it inside and out with a wet sponge.



Also look out for:
If the spring in your doll looks very old/tarnished, please let us know and we will get you a new one.



Also look out for:
If the semicircular bar in your manikin is white, not grey, please also inform us so we can get you a new one.