

I-PAD AED INSTRUCTIONS

In the front pocket of the black bag the AED comes in, you will find a remote control (see below) – you will need this to hand when asking someone to use the defibrillator in front of the other delegates (or demonstrating it yourself).

The voice recording will get stuck at the 'place the pads' stage and you will have to press a button on the remote control – unlike the orange cardiac science AED's which just run through the whole scenario.

S stands for scenario.

Several of the 'S' buttons will allow you to move the recording on to the 'do not touch the patient' voice prompt. **Simone uses S1 as marked on the picture here, interestingly, Josh uses S2!**



The other scenarios buttons do the following:

Scenario - 01

1. A shock is necessary. Deliver the first shock.
2. A shock is necessary. Deliver the second shock.
3. A shock is necessary. Deliver the third shock.
4. A shock is necessary. Deliver the fourth shock.
5. No more shocks required.

Scenario - 02

1. A shock is necessary. Deliver the first shock.
2. A shock is necessary. Deliver the second shock.
3. No more shocks required.

Scenario - 03

1. (Continuous) Shocks are not required.

Scenario - 04

1. A shock is necessary. Deliver the first shock.
2. A shock is necessary. Deliver the second shock.
3. No shock required.
4. A shock is necessary. Deliver the third shock.
5. No more shocks required.

Scenario - 05

1. A shock is necessary. Deliver the first shock.
2. No shock required.
3. A shock is necessary. Deliver the second shock.
4. A shock is necessary. Deliver the third shock.
5. No more shocks required.

Scenario - 06

1. A shock is necessary. Deliver the first shock.
2. No more shocks required.

Scenario - 07

1. A shock is necessary. Deliver the first shock.
2. No shock required.
3. An equipment error has occurred (Error code: 32).

Scenario - 08

1. A shock is necessary. Deliver the first shock.
2. No shock required.
3. A shock is necessary. Deliver the second shock.
4. No more shocks required.

Scenario - 09

1. Continuous shocks necessary.

Scenario - 10

1. A shock is necessary. Deliver the first shock.
2. No shock required.

The function of the other buttons on the remote control:

The buttons at the top are relatively self-explanatory. The team rarely use them – the volume can be helpful, and the pause button can be useful, but it is best to let the device run through continuously when demonstrating..

Power Off

Turns the product on and off.

PAUSE

When this button is pressed, the operation of the product will be paused. When the button is pressed again, the operation of the product will resume. In 'Pause On' status, only the Power Off and Pause buttons operate.

Pads On/Off

Allows the instructor to simulate the pads being placed on the casualty. When the PADS OFF button is pressed, 'Scenario - 06' will be simulated.

Low Bat

When you press this button, a 'low battery' situation will be simulated. The 'low battery' situation simulated is not an actual situation of 'low battery'.

Volume Control


Press the ▲ button, to increase the volume and the ▼ button to decrease it. The default volume of the product is Level 4; control the volume from Levels 1 to 4 with the remote control.

Training Scenario

The buttons S1 - S10 allow the user to select one of the pre-programmed training scenarios.

Language Sel.

Continue to press this button to choose from the 14 languages supported by the product.

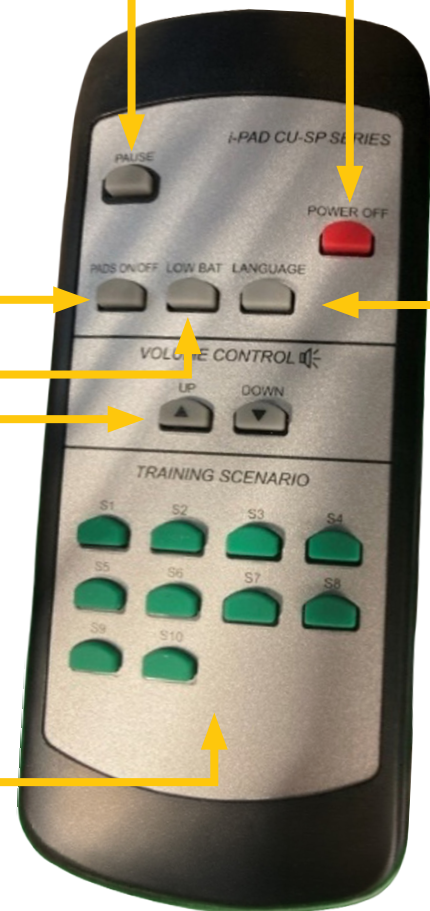

Various controls including Power Off, Pads On/Off, selection of scenario and language/volume control are available in real time through the infrared remote control.

When within 5m of the product, point the remote control at it, ensuring a clear line of sight.

All setting can be controlled with the remote control for easier operation.